


# DR400 User Manual

*patch style Holter recorder  
with event capabilities*

DR400 v5.06

PCPatch v1.06

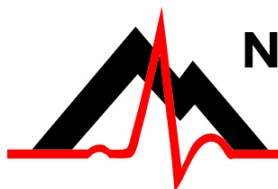

*advancing Holter technology*


**NorthEast Monitoring, Inc.**

141 Parker Street, Suite 200  
Maynard, MA 01754 USA  
[www.nemon.com](http://www.nemon.com)

phone: [+1] 978-461-3992  
fax: [+1] 978-461-5991  
toll free USA: 866-346-5837  
email: [info@nemon.com](mailto:info@nemon.com)



R<sub>X</sub>only

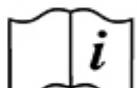

[www.nemon.com](http://www.nemon.com)

**EC REP**  
MediMark Europe SARL  
11 Rue Emile Zola  
38100 Grenoble France



Part number: NEMM048-Rev-D  
Revision: March 20, 2020  
Copyright 2020  
All rights reserved

# Table of Contents

---

**Chapter 1 - About the DR400 ... 3**

- Specifications ... 3
- Intended Use ... 4
- Indications for Use ... 4
- Device Charging ... 4
- LED Display ... 4
- Warning Symbols ... 5
- Instructions to the User About Electrical Interference ... 5
- Patient Leads ... 5
- Storage Capacity ... 5
- Warranty Repairs ... 6
- Operating the Recorder ... 6
- Online help ... 6
- Getting Started ... 6

**Chapter 2 - Recording ECG ... 7**

- Step 1: Fully charge the DR400 ... 7
- Step 2: Start the PCPatch utility ... 7
- Step 3: Prepare patient ... 7
- Step 4: Prepare DR400 ... 8
- Step 5: Start Recording ... 9
- Step 6: Copy Holter file ... 9

**Chapter 3 - The PCPatch Utility ... 10**

- DR400 Settings ... 11
- To View Live Patient ECG ... 12
- Wireless Event Recording ... 13
- Technical Considerations ... 14

**Chapter 4 - Appendices ... 15**

- Appendix A: Maintenance and Care of the Recorder ... 15
- Appendix B: Pacemaker Detection ... 15
- Appendix C: Accessories ... 16
- Appendix D: Extraction of ECG data on 3-channel ... 16
- Appendix E: EMC Information ... 17
- Appendix F: Test Issues ... 20

---

# Chapter 1 -About the DR400

---

The NorthEast Monitoring DR400 patch style recorder can be used as either a Holter monitor and/or a looping Event recorder. The DR400 is designed to facilitate ambulatory cardiac monitoring, on order of a physician, of those patients (including infants weighing less than 10 kg.) who may benefit from such monitoring, including but not limited to those with complaints of palpitations, syncope, chest pains, shortness of breath, or those who need to be monitored to judge their current cardiac function, such as patients who have recently received pacemakers.

Data obtained by monitoring is not analyzed at the time of recording. After recording is complete, Holter data is uploaded to a compatible version of NorthEast Monitoring HE/LX (Holter) Analysis software. For Holter, the DR400 recorder is compatible with 5.2 or newer versions of HE/LX Analysis. For Event, data is sent wirelessly to a location where the LX Event software is used to analyze. The DR400 recorder wireless Event is compatible with LX Event version 2.11 or newer.

## **Specifications**

*The DR400 recorder is not intended to replace real-time telemetry monitoring for patients suspected of having life-threatening arrhythmias and is not for In Vitro diagnostic use.*

### **Physical Specifications**

The DR400 recorder meets the following physical specifications:

- Size: 6.8 cm (length) x 3.9 cm (width) x 1.3 cm (depth)
- Weight: 34 grams (1.2 oz.) with battery

### **Electrical Specifications**

The DR400 recorder electrical specifications are:

- Recording bandwidth: 0.05 to 70 hertz in 180 samples/sec. mode\*
- Operation duty cycle: Continuous
- Data storage format: Sample difference
- Pacemaker sensitivity: 2 millivolts
- Pacemaker pulse duration: 150 to 2,500 microseconds
- Resettable fuses: 0.5 amp

\* When measuring millivolts, results should be within 5% of expected results.

## Environmental Specifications

This equipment is not suitable for use in the presence of a flammable anesthetic mixture with air, oxygen, or nitrous oxide.

The operating range of the device is between 10 and 45 degrees C, between 10 and 95% humidity, and between 500 and 1060 hPa pressure.

Store and/or transport the recorder at temperatures between -20 and 60 degrees C for up to 1 month, and -20 to 30 degrees C for long term; and between 10 and 95% relative humidity, and 500 and 1060 hPa pressure.

## IP44 Ingress Protection

The DR400 recorder has an Ingress Protection Marking of IP44. Dust will not enter device in sufficient quantities to interfere with satisfactory operation. Water jets will not have harmful effect.

## Wireless Specifications

The DR400 recorder is equipped with a wireless Bluetooth transmitter. The Bluetooth specifications for the DR400 are:

- Receive Sensitivity: 95 dBm
- Output Power: 10.5 dBm max
- Link Budget: Up to 105.5 dB
- RX/TX Turnaround: 150 us
- Frequency: 2402 – 2480 MHz in 1 Mhz steps
- Data Rate and Modulation:  
BR: 1 Mbps,  
GFSK / EDR: 2-3 Mbps PSK
- Number of Channels: 79

## Intended Use

- **Holter Mode:** Detection of Arrhythmias, Efficacy of Pharmacological Treatment, and Pacemaker Evaluation.
- **Event Mode:** The event recorder module is a patient activated device designed to record and for diagnostic evaluation of transient symptoms (such as dizziness, palpitations, syncope, and chest pain).

## Indications for Use

1. **Detection of Arrhythmias:** The DR400 recorder is indicated for use in continuous monitoring of cardiac rhythm when intermittent arrhythmia are suspected due to patient symptoms such as palpitations, transient ischemic attacks (TIAs), syncope (fainting), or other such symptoms as determined by the physician.
2. **Efficacy of Treatment:** DR400 recorder is indicated for use to determine whether current pharmacological treatment(s) of known arrhythmia is effective by measuring the frequency and duration of the arrhythmia compared to the frequency and duration prior to treatment.
3. **Pacemaker Evaluation:** The DR400 recorder is indicated for use to evaluate the function of implanted pacemakers to insure that the pacemaker is functioning within prescribed limits.
4. The DR400 recorder is to be used only on the order of a physician.

## Device Charging

The DR400 recorder has an internal rechargeable Lithium battery that is not user replaceable. The recorder comes with a USB cable that attaches to any USB charging source (power plug) for recharging. The battery will take at most, 2 hours to fully recharge between uses.


**Warning: A UL 60950-1 certified charger is recommended for use with the recorder. A non-certified charger may not be safe to use.**

## LED Display

The DR400 recorder has an LED display which consists of a single light that indicates various modes and conditions.

## Warning Symbols

Please note that the recorder is labeled with the following warning symbols:

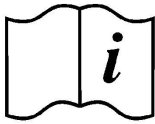

Refer to instruction manual/booklet. Follow instructions for use



Type BF device.



This device contains an internal lithium battery that may be recycled at end of life. This device and all other accessories should be disposed of according to local ordinances.



This product does not contain lead.



Caution: Federal law restricts this device to sale by or on the order of a physician



This device has Bluetooth capabilities.

**Warning:** Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

## Instructions to the User About Electrical Interference

This equipment has been tested and found to comply with the limits for a Class-B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Consult the dealer or an experienced radio/TV technician for help.

This equipment has been certified to comply with the limits for a Class-B computing device, pursuant to FCC Rules. In order to maintain compliance with FCC regulations, shielded cables must be used with this equipment. Operation with non-approved equipment or unshielded cables is likely to result in interference to radio and TV reception. The user is cautioned that changes and modifications made to the equipment without the approval of manufacturer could void the user's authority to operate this equipment.

## Patient Leads

The DR400 recorder is used exclusively with NorthEast Monitoring patch electrodes which are available in 1- and 3-lead configurations.

## Storage Capacity

The patient's ECG data is stored on non-volatile memory internal to the recorder. The internal memory can store up to 470 MB of data.

## Warranty Repairs

The warranty for NorthEast Monitoring products can be found on our web-site at [www.nemon.com](http://www.nemon.com). Contact your dealer or NorthEast Monitoring prior to returning a recorder for repair to determine the warranty period, conditions and exclusions. If your dealer is unavailable, contact NorthEast Monitoring directly.

The recorder can only be serviced or repaired by NorthEast Monitoring or a NorthEast Monitoring authorized representative.

Prior to returning a recorder, you must obtain a return merchandise authorization (RMA) number. This RMA number must be visible on the outside of the packing carton, otherwise, NorthEast Monitoring will refuse delivery. The usable life of the device and accessories are at least long as the warranty period.

## Operating the Recorder

If you require training, such as assistance in setting up, using, or maintaining your recorder, contact NorthEast Monitoring or your dealer. Should the recorder fail to work properly during its useful life or changes its performance, stop using immediately and contact NorthEast Monitoring or your dealer.

**Warning: The recorder cannot be serviced while in use on patient.**

**The DR400 recorder contains no user-serviceable parts. Removing the label or opening the recorder voids the warranty.**

NorthEast Monitoring can be contacted at: [+1]978-461-3992, toll-free in the U.S.A. at 866-346-5837, or email [support@nemon.com](mailto:support@nemon.com).

The patient is not the primary operator, but may be asked to press the EVENT button or transmit events.

Care should be taken when this device is used, especially with infants or small children, as it includes small internal parts that could be a choking hazard. Additionally, the leads could become entangled and could be a strangulation hazard.

## Online help

In addition to the information in this manual, more information and help can be found at our web site, [www.nemon.com](http://www.nemon.com) or by emailing technical support at [support@nemon.com](mailto:support@nemon.com).

The [DR400 Technical Support](#) page on the web-site includes Frequently Asked Questions. The most current version of manuals, the warranty and our software can also be found on our web-site on the "Downloads & Documents" page.

## Getting Started

The DR400 recorder comes with a Quick Start Guide, a USB cable and power plug, and a reusable patient lead. A supply of disposable electrodes may also have been included with your recorder.



Before recording, you should charge your DR400 and download and install the PCPatch utility. More information about installing and using the PCPatch utility can be found in Chapter 3 and on the

DR400 Quick Start Guide.

The PCPatch utility download, a copy of the current manual, the DR400 Quick Start Guide and technical support can be found at [www.nemon.com](http://www.nemon.com).

---

# Chapter 2 -Recording ECG

---

The DR400 recorder has an internal battery and memory. Battery charging is done via a USB cable attached to the power plug. The NorthEast Monitoring PCPatch Utility is used to update settings and download data. In order to do this, the DR400 will be attached to the PC using a USB cable.

## **Step 1: Fully charge the DR400**

**You must fully charge your recorder before its first use.** To charge the internal lithium battery, plug the DR400 recorder into a USB charger (not a PC). While charging, the DR400's LED light will flash orange. Once fully charged, the LED light will flash green every 5 seconds. At this point, detach the DR400 from the charger, and the recorder will stop flashing and go into standby mode.

It takes 2 hours to fully charge the battery from having no charge. A fully charged battery will record Holter for up to 30 days, but we highly recommend that you charge the DR400 between patients. For 24 hour recordings, this will take only about 10 minutes between recordings. For Event recording, the DR400 should be charged every 7 days.

If the DR400 is used frequently, for example, not left in the charger for more than a day or two, it is fine to keep it plugged in between uses. If you use your DR400 less frequently, for example it might be left in the charger for more than a few days, it is best to remove the DR400 from the charger once full charged. You will then want to plug it in again to ensure it is fully charged before your next study, but that should only take a few minutes.

## **Step 2: Start the PCPatch utility**

After charging, but before starting a new recording, plug one end of the USB cable into the DR400 and the other into the PC. Start the PCPatch utility and:




1. ERASE any data on the DR400. This will also update date and time.
2. Enter the NEW Patient ID for the next recording.

*See chapter 3, The PCPatch Utility, for more information about downloading, installing and using the PCPatch utility with your DR400.*

## **Step 3: Prepare patient**

Patient preparation and patch placement is critical to obtain a quality ECG signal. To ensure proper hook-up, follow these steps:

1. There are 3 hook-up options for the DR400:  
Select either a 1-lead electrode patch (1CH) for event or Holter, a 1-lead electrode with the addition of a ECG lead wire (3CH) or a 3-lead patch (3CH)
2. Determine the best location to attach the electrode patch and optional lead wire onto the patient, but do not attach at this time.
3. Prepare the patient's skin. If the patient has hair in any of the site, shave it with a safety razor. Use an alcohol pad and rub the sites briskly until the skin reddens. For best patch adhesion, allow the skin air dry before proceeding.

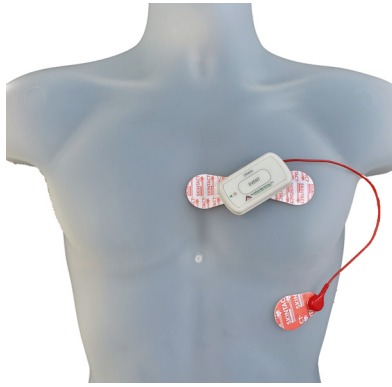

*1-lead electrode patch with optional lead wire (3CH)*



*3-lead patch placement*

### **Step 4: Prepare DR400**


#### **1. Test the DR400 to ensure it is ready for the next patient.**

Do this by removing the DR400 from the USB cable, wait for the flashing to stop, and then quickly pressing the EVENT button. If the light immediately turns orange, this means one of two things: 1) the previous patient's data has not been erased or 2) it has been erased and there is no new Patient ID. You will need to plug the recorder back into the USB and use the PCPatch utility to delete the file and/or enter a new Patient ID.

If you have one, you should insert a USB plug at this time.


#### **2. Attach electrodes to DR400.**

If using the 1-lead electrode patch with the lead wire, attach the ECG lead wire first. The ECG lead attaches to the snap closest to the USB port. Once attached, the clear plastic piece will cover the USB port and be aimed to the right as pictured below:

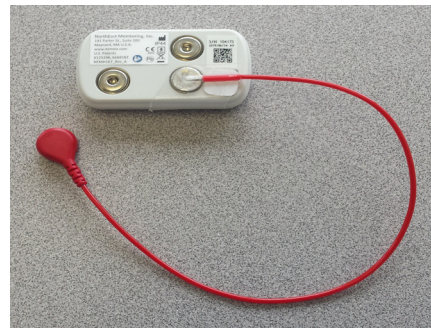

*DR400 with lead wire attached*

Next, attach the 1-lead electrode patch diagonally on the other 2 snaps. Press down firmly so that all snaps are locked in place.

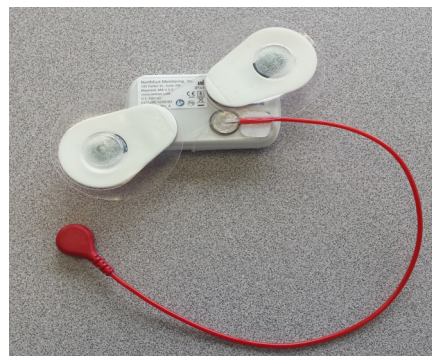

*DR400 with lead wire and adapter for 3CH*



## Step 5: Start Recording

1. Once the electrode patch and optional lead wire are attached to the DR400, remove paper backing from electrodes and press firmly on to patient to attach. Attach the optional lead to the patient once the patch is in place.
2. If you want to observe lead quality at hook-up, start the PCPatch utility now. Once the PCPatch utility finds the Bluetooth USB and is “waiting for DR400”, the utility is ready for you to start the DR400 recorder.
3. To start the DR400 recorder, hold the “EVENT” button down firmly until the green light goes off. At the beginning of a new procedure, the DR400 first shows a solid green light which goes off when recording has started. You can tell it has started when:
  - The DR400 will flashes green for 30 seconds (v5.04) or 60 minutes (v5.06)
  - If the PCPatch is on and the DR400 has been paired with the local Bluetooth, the ECG viewer will automatically open and you will be able to view the ECG.

**Warning:** *If the light on the DR400 turns a solid orange, this means that the previous patient's data is still on the DR400 or the DR400 does not have a Patient ID. When this occurs, you must plug the DR400 into the PC and use the PCPatch utility to save and/or delete the previous patient's data and/or enter a new Patient ID.*

4. Advise the patient that showering is allowed with the DR400 and patch attached, but we do not recommend that the patient go swimming or bathing while wearing either.
5. Instruct the patient on how to use the EVENT button to indicate symptomatic events or activities of interest during the Holter test. Advise them to push the EVENT button briefly.
6. Provide the patient with instructions on how to use and change the patch and recharging instructions for Event recording.

## Wireless Event

For Event recording, the patient must also be sent home with a NorthEast Monitoring Gateway that is set up to transmit events wirelessly and a USB charger with instructions to charge the DR400 about every 7 days. Let the patient know that the DR400 will flash

orange while charging, and then flash green when fully charged.

When the recorder is in Event mode, the recording will restart if the DR400 is erased or the recorder has been plugged in for > 48 hours.

Refer to *Chapter 3, The PCPatch Utility for Event Settings*, and pairing with the Gateway instructions.

## Step 6: Copy Holter file

1. After recording is finished and the patch and lead wire have been removed from the DR400, plug the recorder into the USB cable that is attached to the PC where the PCPatch utility is installed.

**Note:** *For Holter, recording stops once the DR400 is plugged into the USB cable for 2 minutes.*

2. Start the PCPatch utility and copy the flash.dat file (ECG data file). When doing a copy, the utility will automatically find the next open patient directory in HE/LX Analysis.
3. Once copied, you may want to erase the file from the recorder.
4. Remove the DR400 from the PC USB. At this point plug the DR400 into a power source to recharge the battery for its next use.

---

# Chapter 3 - The PCPatch Utility

---



The DR400 uses the PCPatch utility to save ECG patient data to HE/LX Analysis, enter patient IDs, and update settings on the recorder. The utility can also be used to view your patients ECG data live via a Bluetooth USB that is attached to the computer.

## Download and Install the utility


Go to the NorthEast Monitoring web-site at [www.nemon.com](http://www.nemon.com) to download the PCPatch utility. Select either the “Analysis” or “Remote” version, based upon your application.

Once downloaded, install the PCPatch on the computer where the HE/LX Analysis or LX Remote software resides. The default directory, “c:\nm\bin” or “c:\nm\Remote” is where the PCPatch utility will be installed, unless you specify otherwise. If your HE/LX Analysis or LX remote software is located in a different directory, you will need to override the “nm” directory at install. Once installed, you should see a shortcut on your desktop. Click on the shortcut to run the PCPatch utility.

## Using the PC Patch Utility

Plug the DR400 into the USB on the same PC where the PCPatch utility is running. When the recorder shows a solid green light, it is ready to communicate with the PC. When the PCPatch is running will see the main screen., and when the DR400 is found, you can do the following:

- **Preview** - When a patient file is on the DR400, you can view the Recorder S/N, the Patient ID and the date and start time of the recording
- **Copy Flash.dat** - When recording complete, save the patient file to the patient directory. The PCPatch will automatically select the next empty patient directory to save the file to.
- **Erase DR400** - After the patient file is saved, use this to erase the patient file from the recorder. You will need to do this before you can start a new recording
- **New Patient** - After erasing previous patient, enter the Patient ID for the next recording
- **Settings** - Update the Recorder settings and pair with Bluetooth and/or Gateway

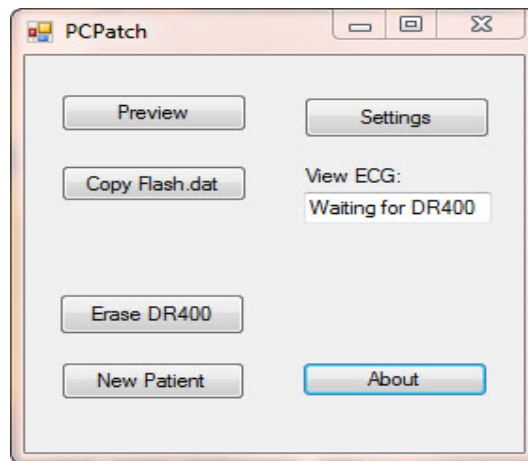

*PCPatch main screen with Bluetooth available*

**DR400 Date and Time.** When the data on the DR400 is erased using the PCPatch utility, the date and time on the recorder will be updated to match the date and time on the PC. After erasing, you can view the date/time on the right-hand side of the PCPatch Settings screen.

## DR400 Settings

Use the PCPatch utility to update the DR400 settings:

**Note:** For DR400 version 5.03, only Holter recording and Bluetooth are enabled. For version 5.04 and greater, Event settings and Gateway are available.

**Recorder Type.** Holter will record up to 3-channels, Event will record 1-channel.

**Days to record.** Stops recording after the number of days. For Holter, up to 14 days and for Event, 30 days is recommended regardless of the procedure length.

**Event Marker.** When “On”, the ECG will be labeled with one second of 6-cycle square wave at the event.

### Event Settings

Are enabled when Recorder Type = Event.

**Pre Event Seconds.** Number of seconds saved before the EVENT button is pressed or an automatic event is detected. Up to 240 seconds allowed.

**Post Event Seconds.** Number of seconds saved after the event. Up to 240 seconds allowed.

### Auto Detect Settings for Event

**Auto Detect** Turn auto detection On/Off for all event types, including AF.

**AF Detect.** Turn Atrial Fibrillation detection On/Off.

**Minimum Time Between AF.** For AF events only. The range of 2 - 90 minutes.

**Minimum Peak AF HR.** Minimum HR that at least 3 beats of the previous 20 must exceed, in order for an event to be called AF. The range is 60 - 120 BPM.

**Brady Limit.** If a heart rate on or below this number is detected, a brady event will be recorded. The range is 20 - 100 BPM.

**Tachy Limit.** If a heart rate on or above this number is detected, a tachy event will be recorded. The range is 50 - 300 BPM.

**Pause Limit.** If no heart beat is detected for at least the pause length in seconds, a pause event will be recorded. The range is 1.0 to 10.0 seconds.

**Separation.** Limits the number of events by type by requiring a minimum amount of time between the same type of event. Applies to all events, except AF. The range is 2 - 20 minutes.

**Update Recorder and Close.** Click to save changes. Close window to make no changes.

### About the DR400

The following items are about the DR400 and cannot be updated via the PCPatch: Serial No. of DR400 recorder, CC (Customer Code), Version number of firmware on DR400, and Build number of firmware on DR400

**Date and Time .** The current date and time are automatically updated when the DR400 is erased using the PCPatch utility.

The screenshot shows the 'settings' window with the following fields and values:

Field	Value
Recorder Type	Event
Days to record	15
Event Marker	On
Pre Event Seconds	30
Post Event Seconds	30
Auto Detect	On
AF Detect	On
Minimum Time Between AF	2
Minimum Peak AF HR	60
Brady Limit(BPM)	50
Tachy Limit(BPM)	120
Pause Limit(seconds)	3.0
Separation(seconds)	2
Gateway	00043e08ce24
Service URL key	nemon
Remote Server URL	nemondb.dyndns.org
Remote Server Port	8000
Carrier APN	11583.mcs
Key ID	4047
Paired Bluetooth USB	000195530c54
Serial No.	104258
CC	0
Version	V5.04
Build	2216
Date and Time	10/09/19 13:31:00
total events	0
remaining	0

Buttons: Update Recorder and Close, Pair Gateway, Pair Bluetooth USB

PCPatch Settings Screen

## To View Live Patient ECG

The DR400 transmits a Bluetooth signal for ECG viewing. When the PCPatch is running and receives the Bluetooth signal, it will open an ECG Viewer screen. You must have a the Bluetooth USB adapter pictured below to view the patient ECG on the PC.




### First, install Bluetooth USB.

Plug the Bluetooth USB into the same PC where the PCPatch utility is installed. It may take a short time for the Bluetooth USB driver to be installed onto the PC. Once


installed and functional, the mode button on the Bluetooth USB will flash green.

### Second, start the PCPatch utility.

Upon starting the PCPatch, you will first see “Looking for BT” while the utility searches for a Bluetooth USB that is physically attached to the PC.

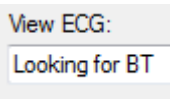

When the PCPatch Utility is running and the Bluetooth USB is found and ready to receive a signal, you will see the message, “Waiting for DR400”.

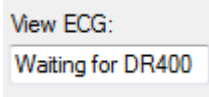

**Note:** The PCPatch utility only searches for the Bluetooth USB at start up or while attempting to pair with a DR400.


**Third, pair the DR400 to Bluetooth USB.** Before you can view live ECG with a particular DR400, you will first need to pair the DR400 with your Bluetooth USB. On the PC where the Bluetooth USB is installed, plug in the DR400, open the PCPatch, and go to the Settings screen. Click on the “Pair Bluetooth USB” and the PCPatch will attempt



to pair the two. Once paired, you will see the USB number on the Settings screen. The last 5 numbers should match the Serial Number of your Bluetooth USB. The button is only enabled when a Bluetooth USB was found when the PCPatch utility started.

## Using PCPatch to View ECG

With the DR400 ready to record and attached to the patient, start the PCPatch application. Once the utility shows “Waiting for DR400”, hold the EVENT button down to start the recording. Once the green light turns from solid to flashing, recording has begun and transmits Bluetooth for about 15 seconds. After a brief pause, the PCPatch utility should open the window to view ECG.

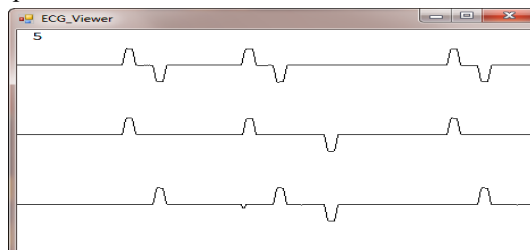


ECG Viewer (test data shown)

If recording has already started and you wish to view the ECG, hold the button down until it starts flashing green rapidly, and then release.

## Troubleshooting

No Bluetooth “No BT” message:

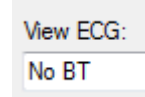

- If the Bluetooth USB is installed on the same computer, and the PCPatch fails to see it, you can try to restart the PCPatch so that it searches again. The PCPatch only searches for the Bluetooth USB at start up.
- Unplug and reinsert Bluetooth USB. Check to make sure that the mode light is flashing green. Restart the PCPatch utility to restart the search.
- Drivers for the Bluetooth USB were included with your Windows Operating System. But if you plug in the Bluetooth USB for the first time and the PC is unable to see it at all (the mode light on the USB is not flashing green), you may need to locate and reinstall the device drivers.

“Waiting for DR400” message:

If you see this message and the DR400 appears to be transmitting but you have no ECG screen, perhaps the DR400 is not paired with the Bluetooth USB that is plugged into the PC. If you want to fix this now, you will need to remove the DR400 from the patch electrode and plug it into the USB cable to pair it with the Bluetooth USB from the Settings screen.


## Wireless Event Recording

Event recording on the DR400 is done wirelessly, via a NorthEast Monitoring Gateway and the Socket software. Refer to the *Gateway - Socket Technical Manual, NEMM046*, for more information on how to set up and run the Wireless feature.

### The Gateway Settings

The Gateway field on the Settings screen shows which Gateway has been paired with the DR400.

The Service URL key, Remote Server URL, Remote Server Port, Carrier APN, and Key ID are all required to transmit successfully.

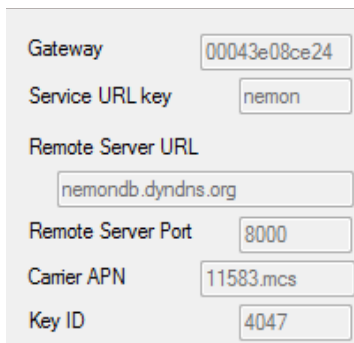

Gateway	00043e08ce24
Service URL key	nemon
Remote Server URL	nemondb.dyndns.org
Remote Server Port	8000
Carrier APN	11583.mcs
Key ID	4047

If any of these items are blank or incorrect, you can update these by putting your URLKEY.dat file into the bin directory and pairing the Gateway again. If your URLKEY.dat is incorrect or you do not have a URLKEY.dat file, contact NorthEast Monitoring or your distributor for assistance.

**Important:** A *URLKEY.dat* file must exist in your directory in order to pair with a new Gateway. If the file does not exist, the Pair Gateway button will not be enabled.

### Recording an Event Procedure

The setup and recording procedure are the same as Holter and are described in the previous chapter. While recording in Event mode, the patient should charge the DR400 about every 7 days. Remind the patient that he recorder will flash orange while charging, and then flash green when fully charged. Up to 30 days of event recording is possible in a single event procedure.

As with Holter, the patient can record an event by pressing the EVENT button quickly. The recorder will flash green when recording an event. Flashing

will also occur when the recorder is saving an automatically generated event.


Once an event has been saved, the DR400 will attempt to contact the Gateway every 20 minutes for the first hour, and then once an hour after that. Once contact is made with the Gateway, all events will be transmitted and the process will begin again when a new event is saved.

### Transmit Events Manually

With the Gateway present and the Socket software running, hold the EVENT button down for about 10 seconds. Once the DR400 starts flashing rapidly, release the button, and the DR400 will transmit to the paired Gateway. This can be done during of after the procedure has stopped.

### Events on returned DR400

After the event procedure, the Settings screen will show you the total number of events that were saved as well as any that are still remaining and not transmitted. If you would like to save the remaining events, you will need to plug in the Gateway and transmit the events manually as described above.

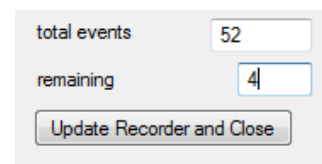

total events	52
remaining	4
Update Recorder and Close	

## Technical Considerations

### LX Remote Installations

A special version of the PCPatch utility must be used with LX Remote to copy the patient files. You must install the PCPatch Utility on the PC where the LX Remote desktop application exists. The installation will default to the c:\nm\Remote directory.

Once the DR400 file is copied to the LX Remote directory, you should open the Patient Information window in LX Remote and enter the patient information before sending the file.

### Network Installations

For Holter, if the PCPatch utility is being installed on a PC that does not have HE/LX Analysis installed, you will need to do the following to allow the user to save patient files to HE/LX Analysis:

1. Ensure that the local PC has read/write access to the HE/LX Analysis patient directory on the server
2. On the local PC, create a directory labeled "c:\nm\bin"
  - Copy the h4w.ini file from a PC that has HE/LX Analysis installed and paste it into the local directory. (The h4w.ini file most likely can be found in the c:\nm\bin directory.)

Using Windows, test to ensure that the path to the Patient Directory works for the PC where PCPatch is installed.

---

# Chapter 4 - Appendices

---

## **Appendix A: Maintenance and Care of the Recorder**

Clean the outside of the recorder with a damp soft cloth between uses; use water and a non-abrasive liquid soap, as required. DO NOT use any abrasive cleaners, such as acetone, on the outside of the recorder.

Disinfect as needed, following instructions from your infection control department. Sani-Cloth germicidal surface wipes are recommended.

Sterilization is not needed.

Do not submerge the recorder in water.

At the end of their useful lives, all NorthEast Monitoring Inc. products should be disposed of following local ordinances.

## **Appendix B: Pacemaker Detection**

The recorder has a built-in pacemaker detection capability. This was designed to overcome the problems inherent with the analysis of Holter recordings from patients with pacemakers.

A pacemaker is designed to initiate cardiac conduction by stimulating a spot on the myocardium with a pulse of 1-4 volts and a duration of typically 250 to 2,000 microseconds. When this pulse is seen at the surface recording electrodes it is significantly attenuated. For patients with a unipolar electrode configuration, the signal at the surface may range from under 50 to over 200 millivolts. When a bipolar lead configuration is used, the signal is typically much lower and is in the range of 3 to 50 millivolts. Especially with the bipolar leads, the signal size is dependent on the positions of the pacemaker lead and the surface electrodes.

The amplitude of the signal being referred to here is not the size of the "spike" commonly seen on an ECG cart or bedside monitor. Since the duration of the pulse is short compared to a QRS complex, normal ECG recorders will greatly attenuate the signal; in some cases it cannot be seen at all. Also, some ECG recorders have devices which enhance the pace pulse to insure that it will be displayed. Only very wide bandwidth recorders as are sometimes used in an electro-physiology study will show the unmodified full amplitude of the pulse.

The recorder has the wide bandwidth ECG amplifiers necessary to pass the pacemaker pulse. Since the pulse would still be too short to be recorded in a reliable manner at any practical sampling rate for Holter recording, the pulse is detected by the recorder. The time of the pulse is then digitally stored along with the Holter ECG data. When the data is analyzed, the pacemaker pulse is displayed and used for the analysis.

At recording time, it is desirable to have the recorder be as sensitive to the pacemaker pulse as possible so pulses will not be missed. A conflicting requirement is that there should be as few false pacemaker detections as possible.

False pacemaker detections are primarily caused by electrical events. Any external electrical signal that is coupled to the patient electrodes which looks like a pacemaker pulse will of necessity be stored by the recorder. The most common form of electrical signal that can look like a pacemaker signal is an electrostatic discharge (ESD) or "spark." These happen very frequently in dry weather but also occur, at a lower rate, under humid conditions.

Fortunately most ESD spikes as seen at the patient electrodes are of shorter duration or of lower amplitude than the real pacemaker pulses. While there is no absolute limit to the size or duration of the ESD pulses, the recorder ignores all pulses that are less than 150 micro-seconds long or are less than two millivolts in size.

As pacemakers are normally programmed to a pulse width greater than 200 microseconds, this does not cause a loss of detection. The requirement that the pacemaker pulse be at least two millivolts in size is not a common problem.

## Appendix C: Accessories

Contact your distributor to purchase accessories for the DR400.

**NEMEL001** . 3-Channel Hook-Up Kit (Adapter patch with 2 electrodes, plus extra electrode for 1-lead wire)

**NEMCA160** . 12" Reusable 1-lead wire (for use with NEMEL001 for 3-channel recording)

**NEMCA161**. 16" Reusable 1-lead wire (for use with NEMEL001 for 3-channel recording)

**NEMEL003**. 3-lead electrode patch

**NEMP00513**. Sena Parani Mfg PN SD1000U: Bluetooth USB Adapter for Serial Port

## Appendix D: Extraction of ECG data on 3-channel

It is possible to retrieve the raw ECG files from the Holter files. For all 3-channel data, the process results in three files, one for each channel. Each file is then in the form of a binary file consisting of 16-bit words (little endian) with each word representing one sample. The sampling is at 180 samples per second. The data is scaled so that the least significant bit has a value of 12.5 uv. If a pacemaker pulse was detected, the sample at the time of detection will be replaced by the value 0x8000.

To generate these files, first analyze the data (actually the flash.dat) from the recorder using any compatible version of the LX Analysis program. At the completion of this there will be a file "datacard.dat" in the patient directory. The full path is by default:

```
c:\nm\pat\xx\datacard.dat
```

where xx is the number of the patient dataset. This can be seen in the "No. and Directory" columns of the "File->open/new" display.

Then, change the directory to c:\nm\bin and on a single command line, run the following command using the following 5 arguments:

```
unpackc d1 f1 f2 f3 0
```

- where d1 is the path to the source datacard file, for example,  
d1 = c:\nm\pat\xx\datacard.dat
- f1, f2 and f3 are the resultant binary destination files, for example:  
f1 = c:\nm\pat\xx\flashc0.dat  
f2 = c:\nm\pat\xx\flashc1.dat  
f3 = c:\nm\pat\xx\flashc2.dat

The result will be the three files in the patient directory xx described previously. The files are flashc0.dat flashc1.dat and flashc2.dat which are for channel 1,2 and 3 respectively. If desired, the destination paths for this command can be any other path but spaces are not allowed in the path or file name.




## Appendix E: EMC Information

Attention should be paid to the following EMC information prior to installing or using the Northeast Monitoring DR400 Recorder device.

- Portable and mobile Radio Frequency (RF) communication equipment may interfere with the operation of the device.
- The device has been tested and found to comply with IEC/EN 60601-1-2.
- Computers, cables and accessories not tested to 60601-1-2 may result in increased emissions or decreased immunity of the device.
- Verify normal operation if utilizing the device adjacent to or stacked with other electrical equipment.

Guidance and manufacturer's declaration – electromagnetic emissions		
The Northeast Monitoring DR400 Recorder is intended for use in the electromagnetic environment specified below. The customer or user of the Northeast Monitoring DR400 Digital Recorder should ensure that it is used in such an environment.		
Emissions Test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	Northeast Monitoring DR400 Digital Recorder uses RF energy only for its internal function. Therefore, its RF emissions are not likely to cause any in nearby electronic equipment.
RF emissions CISPR 11	Class B	Northeast Monitoring DR400 Digital Recorder is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage Fluctuations/flicker emissions IEC 61000-3-3	Not applicable	

<b>Guidance and manufacturer's declaration – electromagnetic immunity</b>			
The Northeast Monitoring DR400 is intended for use in the electromagnetic environment specified below. The customer or user of the recorder should ensure that it is used in such an environment.			
<b>Immunity test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment – guidance</b>
Electrostatic discharge (ESD) IEC 61000-4-2	± 6kV contact ± 8kV air	± 6kV contact ± 8kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	Not applicable. No cables exceed 3 meters	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	Not applicable. Northeast Monitoring DR400 Recorder is battery powered.	N/A
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 5% $U_T$ (>95% dip in $U_T$ ) For 0,5 cycle  40% $U_T$ (60% dip in $U_T$ ) For 5 cycles  70% $U_T$ (30% dip in $U_T$ ) for 25 cycles  < 5% $U_T$ (>95% dip in $U_T$ ) for 5 s	Not applicable. Northeast Monitoring DR400 Recorder is battery powered.	N/A
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE $U_T$ is the a.c. mains voltage prior to application of the test level.			

<b>Guidance and manufacturer's declaration – electromagnetic immunity</b>			
The Northeast Monitoring DR400 Recorder is intended for use in the electromagnetic environment specified below. The customer or user of the recorder should ensure that it is used in such an environment.			
<b>Immunity test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment – guidance</b>
<p>Conducted RF IEC 61000-4-6</p> <p>Radiated RF IEC 61000-4-3</p>	<p>3 Vrms 150 kHz to 80 MHz</p> <p>3 V/m 80 MHz to 2,5 GHz</p>	<p>3 V</p> <p>3 V/m</p>	<p>Portable and mobile communications equipment should be used no closer to any part of the Northeast Monitoring DR400 Recorder, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p><b>Recommended separation distance</b>  <math>d = 1.2\sqrt{P}</math></p> <p><b><math>d = 1.2\sqrt{P}</math>    80 MHz to 800 MHz</b></p> <p><b><math>d = 2.3\sqrt{P}</math>    800 MHz to 2.5 GHz</b></p> <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,<sup>a</sup> should be less than the compliance level in each frequency range.<sup>b</sup></p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> <div style="text-align: center;">  </div>
NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies			
NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a	Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the recorder is used exceeds the applicable RF compliance level above, the recorder should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or recorder		
b	Over frequency range 150 KHz to 80 MHz, field strengths should be less than 3 V/m.		

<b>Recommended separation distances between portable and mobile RF communications equipment and the Northeast Monitoring DR400 Digital Recorder</b>			
The Northeast Monitoring DR400 Digital Recorder is intended for use in the electromagnetic environment in which radiated RF disturbances are controlled. The customer of the user of the recorder can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the recorder as recommended below, according to the maximum output power of the communications equipment.			
<b>Rated maximum output power of transmitter W</b>	<b>Separation distance according to frequency of transmitter m</b>		
	<b>150 KHz to 80 MHz <math>d = 1.2\sqrt{P}</math></b>	<b>80 MHz to 800 MHz <math>d = 1.2\sqrt{P}</math></b>	<b>800 MHz to 2.5 GHz <math>d = 2.3\sqrt{P}</math></b>
0.01	0.12	0.12	<b>0.23</b>
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.			
NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.			
NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

## Appendix F: Test Issues

1. PCPatch date & time do not reflect local OS Regional settings. The date and times will be correct when you save the recording however.
2. PCPatch erase file error - The PCPatch utility does not recognize that the file on the DR400 is erased and does not allow the user to enter a Patient ID. To fix, close PC Patch utility and remove DR400 from plug. Restart PCPatch and plug in DR400 and try again.
3. Although event files can be copied to the HE/LX software, only the first 7 days of the recording are viewable.