

DR181 Hook-Up Guide for OxyHolter

5-Electrode, 2-Channel for OxyHolter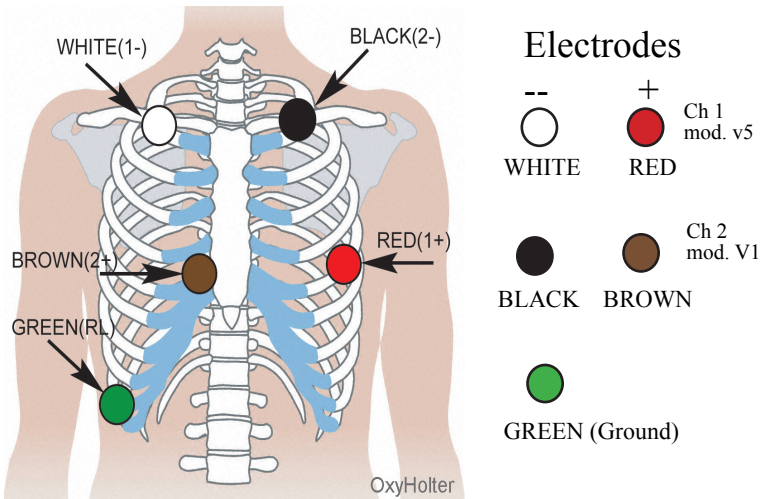


Patient Preparation

- Select area over bone, avoiding muscle areas and breast tissue.
- Clean electrode sites aggressively with alcohol and use a razor to remove hair.
- Abrade electrode sites with a scrub pad or gauze. Allow sites to dry before applying electrodes.
- Firmly snap electrodes to the lead wires.
- Apply electrodes to appropriate sites. Press adhesive border firmly for consistent adhesion.
- To help alleviate tension, loop and tape lead wires down.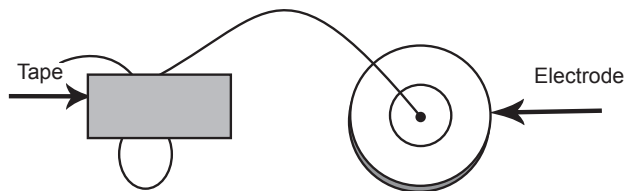


How to attach the probe and finger wrap

1. Remove the paper backing from the finger wrap
2. Attach the oximetry probe with cable to its sticky side, lining up the two bumps on the probe with the cutouts in the wrap
3. Position the patient's finger on the probe. Be sure to line up the tip of the finger with the center line on the wrap
4. Attach the wraps to the patient's finger in the following order:
 - a. Attach the two small side wings
 - b. Fold the wrap and probe back over the fingernail
 - c. Wrap the third small wing
 - d. Wrap the long wing around the finger
5. Use tape around the base of the patient's finger to secure the cable to the finger. Apply additional tape to hand to secure cable.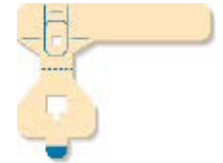

Finger Wrap

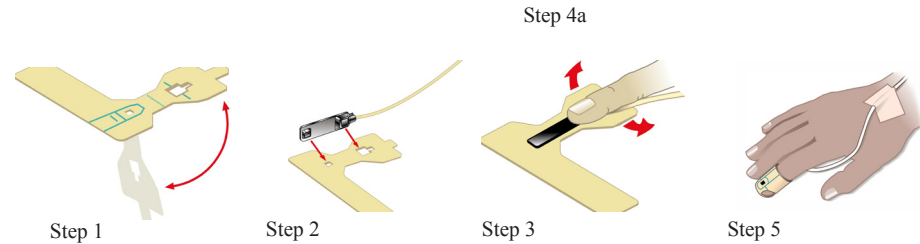MediMark Europe, SARL
11 rue Emile Zola
BP 2332
F-38033 Grenoble CEDEX 2
France

NorthEast Monitoring, Inc.

141 Parker Street, Suite 111
Maynard, MA 01754 USA
www.nemon.com

phone: [+1] 978-461-3992
fax: [+1] 978-461-5991
toll free USA: 866-346-5837
email: info@nemon.com

DR181 Quick-Start Guide for OxyHolter

TO START RECORDING:

1. Attach cable to recorder and then hook-up the patient leads and oximetry cable as shown on reverse.
2. Insert a formatted SD card and 2 AA batteries into the recorder. Erase SD card memory if prompted to do so. The screen will display date, time, battery strength, "DR181", software version and "NorthEast Monitoring". Press Next to continue to main menu.
3. If desired, adjust Settings by pressing the button on the keypad. In Settings, press More to see additional setting options.
4. When settings are correct, go back to the main menu by pressing the Prev (Previous) button.
5. Press New Patient screen to verify or enter Patient ID. At least one character must be entered for Patient ID. Press Next when finished entering Patient ID.
6. You will now see lead quality for 2 channels. Lead quality, or impedance, should be 4 or 5 for best results. Adjust leads if required.
7. For SpO₂ you will should see an oximetry value. If no value is present or if recorder reads "No cable" or "No sensor", be sure that you are using the Oximetry cable and sensor, and adjust the oximetry sensor until a reading is visible.
8. Press "Record" to begin recording.

AFTER RECORDING:

To remove card after recording, gently push inward to release. Never pull card out of slot, as it may damage the recorder.

NOTE: If you want to record a new patient and less than an hour has gone by since you finished your last, the recorder may try to continue recording. You can interrupt the 5 second countdown by pressing the 5-left hand buttons starting at the top. Erase the SD Card and you can now record a new patient.

MESSAGES:

Battery LOW or FAILURE– Replace both batteries before starting recorder.

Erase SD Card? YES/NO - If the SD card has been used for a previous patient, you will need to erase now. If the card should not be erased as it has ECG data that you do not want to lose, you will need to remove it and put in another formatted SD card.

SD Card is full - Recording has stopped because memory is full.

SD Card errors - SD Cards must be formatted using HE/LX Analysis software. Also, the SD Card must have a flash.dat file on it to record. Refer to your HE/LX Analysis Operator's Manual for details on SD Card errors and how to format the card.

SETTINGS:

Time and Date - Toggle between Date and Time in the lower right-hand section of screen.

Language - Choose from multiple languages

Display - Choose between Normal and Inverted displays

Lock Settings - Locks settings on the recorder

Contrast - Adjusts LCD screen contrast

General Settings:

Resolution - Choose between High and Normal resolution

Mark Diaries - Choose whether to mark diary entries on ECG with a 6-cycle square wave

Record Type - Choose **Oximetry** for OxyHolter recording.

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